



Superactive LED Infinity Tunnel 3 Shape

Instruction Manual

For optimal results, please read and follow
instructions carefully
Please keep for future reference



WARNING:
CHOKING HAZARD – Small Parts
Not suitable for children under 3 years.



3+ years

experia
innovation in every sense

Congratulations on the purchase of a Superactive LED Infinity Tunnel 3 Shape!

The LED Infinity Tunnel has been designed specifically for people with special needs and should, if used in accordance with these guidelines, offer years of switching learning and fun!

The LED Infinity Tunnel offers endless opportunities to develop numerous skills including colour recognition, cause and effect, and many more!

Please ensure you read and follow these Instructions carefully before installing or operating the unit. Please keep this manual for future reference.

Heed all warnings on the product and within this manual. If upon receipt of this product, the user suspects that the product is defective, and/or requires service or repair due to malfunction or damage to the product, please contact the manufacturer immediately. Please be aware that improper installation and/or use or misuse of this product may result in injury, death, or adverse reactions to its user. This product is intended to be installed and/or use by a trained professional. Failure to comply with these precautions or with specific warnings elsewhere in this manual violates safety standards of design, manufacture, and intended uses of the product.

Features

- Ergonomic design
- Completely silent
- Latest LED technology
- No lamp change – LED
- Cool running
- Hundreds of shades of colours
- Low Voltage – transformer supplied
- Easy upgrade/maintenance
- Durable
- Easy to use
- 3 Year manufacturers guarantee (Collect and Return)
- 8 switch controller

Pack contents

- 1 x Superactive LED Infinity Tunnel 3 Shape
- 1 x Low Voltage Transformer
- 1 x Controller
- 1 x USB Lead
- 1 x Qi charging Pad
- 1 x USB Power Supply

Please note: Only use the transformer supplied. Use of a different transformer may result in damage to the product and void the warranty. For warranty information please see separate warranty card supplied.

Getting Started

1. Place the Superactive LED Infinity Tunnel on a flat solid floor or mount securely on a wall.
2. Connect the round XLR connector (silver connector) from the Low Voltage transformer to the socket on the side of the unit.



3. Connect the standard mains plug to a mains outlet.
4. The Infinity Tunnel will now illuminate and will scroll through the many colours.
5. Now it's ready to use.

Charging the Switch box

1. Plug the USB lead into the wireless charging pad
2. Plug the other end of the USB into the correct power supply
3. Plug the power supply into the mains socket outlet and switch on at the mains. The charging pad LED should now light up
4. Position the controller over the charging pad so the charging pad is underneath the red button.
5. The charging light on the controller should light up when it is correctly positioned over the charging pad
6. Leave the controller charging up until the LED changes colour.

Cleaning and Maintenance

The unit should be cleaned with a damp soft cloth. A non abrasive cleaning agent may be used for stubborn stains. Ensure no liquid enters the base of the unit.

The unit has been designed for use by people with special needs, however this does NOT mean it is unbreakable. You must ensure the unit is not hit with anything and that no one pulls or pushes the unit.

For best results remove the batteries from the switch controller if it is not being used for a prolonged period.

Troubleshooting

Q. The unit doesn't work.

A1. Is it switched on? If so,

A2. Has the unit scrolled through its colours? If so,

A3. Have you pressed the button on the controller? If so,

A4. Is the controller fully charged? If so,

A5. Please contact us for further assistance.

Troubleshooting the wireless controller

Q. My controller doesn't work

A1. Has the controller been charged using the wireless charging pad? If so

A2. Did the LED light up on the charging pad? If so

A3. Was the controller positioned over the charging pad in the correct position with the charging pad underneath the red button? If so

A4. Did the LED light up on the controller when positioned correctly? If so

A5. Are you using the controller with the correct product? If so

A6. Are the coloured buttons "clicking" when pressed? If so

Please contact Experia headquarters as details on the final page of this document.

Contact Details

For USA please email support@experia-usa.com

For Canada please email support@experia-canada.com

For UK and rest of the world email support@experia.co.uk

Telephone: 0044 (0) 845 644 0977.

Please note: email is our support teams preferred communication and this takes precedence over telephone calls.

QUESTIONS & IDEAS?

If you have an idea on how to improve this product or want to see your product idea realised, please email: info@experia.co.uk.

All ideas are presented to our OT, technical and finance department and if all procedures are passed we will contact you to discuss your idea further.

We can't promise to develop every product but if it's a great idea, we'll try!

Manufactured proudly in the UK by:

Experia

Acorn Phase III, High Street, Grimethorpe, South Yorkshire, UK, S72 7BD



© Copyright 2015 Experia All rights reserved Experia. Experia is a registered trademark.